


TREATMENT PROCESS

1. Contact handpiece on epidermis.
2. Insert MICROELECTRODE upon contacting handpiece. (Possible to control the length of MICRONEEDLE)
3. Produce stimulation by MICROELECTRODE.
4. Give an momentary irradiation of HIGH RF ENERGY on treatment area.

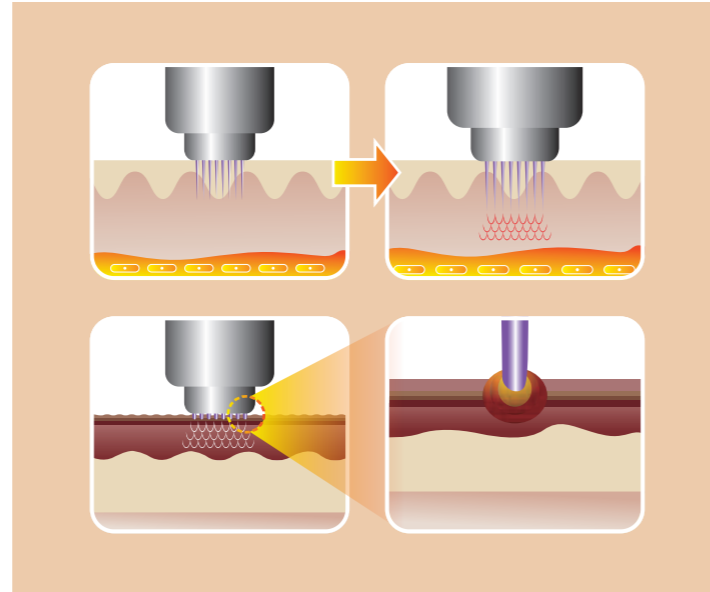

ANTIGE

THE ANTIGE is a promising technique for the treatment of dermis area

Electrode tip is an accessory for **ANTIGE**



SPECIFICATION

TYPE	Electro-surgical Device
MODE	Auto / Manual (Single / Double)
NEEDLE TYPE	Invasive Electrode:36 Pin, Non-Invasive Electrode:36 Pin
DEPTH LEVEL	0.5mm ~ 4.0mm
OUTPUT LEVEL	Single Shot:Level 1~Level 50 Double Shot:Level 1~Level 100
REPEAT TIME	0.5sec ~ 3.0ms, OFF
FREQUENCY	1MHz / 2MHz
ELECTRICAL	100~240VAC, 50Hz/60Hz
DIMENSION	400(W) x 500(D) x 1050(H)mm
WEIGHT	22kg
SMART MODE	Wrinkle, Tightening, Lifting, Nasolabial Fold, Fat
NEEDLE TYPE	Insulation Tip (Abaltive / Non-Abaltive) Non-Insulation - Only ablative hyperhidrosis



ANTIGE

The Latest Fractional micro Electrode RF Tecnique
Optimizing the strong point for RF Tecnique

What is the **ANTIGE**

Dermatology treatment has long attempted to find the surgical procedure equipment that can realize the merit of INVASIVE treatment effect simultaneously. As part of this effort, combination of advantages from the treatment principle of MICRONEEDLE STAMP and THERMAGE (known to be epitome of non-operative FACIAL-LIFTING treatment utilizing high frequency) leads to the realization of MFRF(MINIMALY FRACTIONAL RADIO FREQUENCY), new concept of treatment which has been attempted to resolve the various problems of epidermis and dermis.

Antige acts very greatly to treat various lesions of DERMIS that has been believed to be a limit to the laser treatment. In addition, this equipment is TOTAL FACIAL CARE SYSTEM which can treat safely without impairment of epidermis with the aim of DERMIS COAGULATION.

3 Types of Electrodes for **ANTIGE**

36-pin (ANI-36)	36-pin (ANT-36)	36-pin (ANN-36)
Ablative Electrode (Insulated)	Touch Electrode (Non Ablative Electrode)	Non-Insulated Ablative Electrode (Effective for armpit smell treatment)
		

Strong points and Features of **ANTIGE** treatment

1 Sophisticated and fast treatment of convenience

Depth of penetration can be precisely adjusted in 0.1mm units from 0.5mm to 4.0mm.
;It can adjust the amount of energy - single/double and 1Mhz / 2Mhz.

2 Precise treatment

Needles are available to treat various areas. It facilitates the treatment on both wide and narrow areas.

3 Safe energy transmission system

Insulated Electrodes tip can transmit the energy accurately on dermis area without damaging epidermis.

4 Minimize pain for treatment and no down time

Reduces the damage of epidermis as much as possible by minimizing the diameter of Electrodes.

5 Smart mode

Stored Protocol increase the convenience for users.



Applications & Wonderful Effects of **ANTIGE**

